

Defibrillator Instructions

Use the defibrillator when someone is:

- Unresponsive
- No pulse
- Limited breathing or none

Defibrillate before doing CPR

1. Call 911
2. Grab AED (defibrillator) – note high-pitched alarm goes off when cabinet door is opened.
Place next to head
Pull green handle
Listen to voice prompts
Cut off clothing (scissors provided in AED kit)
3. Place pads as indicated on pad itself (See drawing below)



4. Everyone stand back from person, not touching them, while ECG and/or shock is taking place
5. Machine takes ECG, tells if & when to shock
6. CPR
7. EMS takes the defibrillator unit to the hospital
8. We need to send someone to hospital to retrieve the unit.
9. Un-witnessed arrest, do CPR first, then AED